



The UltraSonic Cleaner

To Reduce Processing Time

With the UltraSonic Cleaner, you can shorten the time it takes to convert an extracted tooth into bone graft by 5 min. (40%).



STEP 1:
Make sure the mixing dish with the dentin graft is completely saturated with the Dentin Cleanser.



Close the lid of the mixing dish firmly so that water from the Ultrasonic unit does not penetrate the dish.





STEP 3: Place one half cup of tap water into the basin (keep the plastic basket in the unit).



STEP 4:
Place the dish on top of the plastic basket in the UltraSonic basin.



STEP 5: Set the timer of the unit to 280 seconds by pressing the 'timer' button until 280 seconds are displayed.

and press the 'ON' button.

Once 280 seconds have passed, the unit will stop on its own.

Close the UltraSonic cover





STEP 6:

Remove the mixing dish from the UltraSonic Cleaner and continue the process by dehydrating the Dentin Cleanser with a sterile cotton gauze and adding the Dentin Wash (PBS) for 3 minutes and then dehydrating it with a new sterile gauze.

Copyright © 2015 by KometaBio Inc. All rights reserved.

All intellectual property rights in this publication are owned by KometaBio Inc. and protected by applicable copyright laws and international treaty provisions.